English

# BARD® Urological Catheter

**Warning:** Do not use ointments or lubricants having a petrolatum base. They will damage latex.


Visually inspect the product for any imperfections or surface deterioration prior to use.

- Use luer tip syringe to inflate with stated mL of sterile water.
- OR
- For pre-filled products, remove clip and squeeze reservoir to inflate with stated mL of sterile water.

For urological use only.

To deflate catheter balloon: Gently insert a luer slip tip syringe in the catheter valve. Never use more force than is required to make the syringe "stick" in the valve. Allow the pressure within the balloon to force the plunger back and fill the syringe with water. *If you notice slow or no deflation, re-seat the syringe gently.* Use only gentle aspiration to encourage deflation if needed. Vigorous aspiration may collapse the inflation lumen, preventing balloon deflation. If necessary, contact adequately trained professional for assistance, as directed by hospital protocol.

Should balloon rupture occur, care should be taken to assure that all balloon fragments have been removed from the patient.

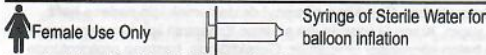Refer to Direct Unit Label for Product Content and Gender Specific Use where applicable.

**LATEX** Contains or Presence of Natural Rubber Latex.  
**Caution:** This Product Contains Natural Rubber Latex Which May Cause Allergic Reactions.

- Single use only. Do not reuse.
- Do not resterilize

This is a single use device. Do not resterilize any portion of this device. Reuse and/or repackaging may create a risk of patient or user infection, compromise the structural integrity and/or essential material and design characteristics of the device, which may lead to device failure, and/or lead to injury, illness or death of the patient.

**LOT** Lot Number      **REF** Reorder Number      Use by Date

Caution, consult accompanying documents.      Consult instructions for use.

Storage: Store catheters at room temperature away from direct exposure to light, preferably in the original box.  
 Keep away from sunlight      Units      Do Not Use if Package is Damaged.

**STERILE R** Sterilized by radiation      OR      **STERILE EO** Sterilized by Ethylene Oxide

Refer to Direct Unit Label for Sterilization Method Utilized

BARD, BARDEX, BARDIA, BIOCATH and LUBRICATH are trademarks and/or registered trademarks of C. R. Bard, Inc.

**Warning:** After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable laws and regulations.

## BARD

**Manufacturer:**  
 C. R. Bard, Inc.  
 Covington, GA 30014 USA  
 1 800 526 4455  
 www.bardmedical.com

**EC REP** BD Switzerland Sarl  
 Terre Bonne Park - A4  
 Route de Crassier 17  
 1262 Eysins, Switzerland

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