en

**OBSERVE UNIVERSAL PRECAUTIONS ON ALL PATIENTS  
REPORT NEEDLE STICKS IMMEDIATELY AND FOLLOW ESTABLISHED PROTOCOL.**

- Percutaneous puncture with a contaminated needle may lead to serious illness such as hepatitis, HIV (AIDS), or other infectious diseases.
- Resheathing needles is hazardous.

**NON-PYROGENIC - STERILE:**

Unless package has been opened or damaged.

**DO NOT RESTERILIZE.**

As with any spinal procedure, to help avoid needle breakage, **DO NOT ATTEMPT TO STRAIGHTEN A BENT NEEDLE.**

Care should be taken to avoid needle damage. Repeated repositioning may increase the risk of needle breakage. Observe needle carefully during procedures.

Remove spinal needle and introducer needle together. Replace with a new needle.

**ENSURE SYRINGE IS SEATED FIRMLY IN NEEDLE HUB.**

To be used by properly qualified and trained health professionals.

**CAUTION:** Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

Re-use may lead to infection or other illness/injury.

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For Switzerland: Imported by: Becton Dickinson AG, Binmingerstrasse 94, 4123 Allschwil, Switzerland

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BD and the BD Logo are trademarks of Becton, Dickinson and Company. © 2022 BD. All rights reserved. bd.com

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Becton Dickinson S.A., Camino de Valdeoliva, s/n, 28750 San Agustín del Guadalix, Madrid, Spain
Becton Dickinson S.A., Camino de Valdeoliva, s/n, 28750 San Agustín del Guadalix,