REVISION RECORD			
CURRENT REVISION PREPARED BY: <i>P. Kumar</i>		ARTIST: <i>M. Ferreira</i>	
Rev.#	Description	ECO #	Doc. Rev. Date
02	<i>Updated Revision record and Title block template. Added Product description as per registration certificate (all SKU's). Corrected Russian translation text for “Non-Pyrogenic”. Added the caution statement. Updated SAP#.</i>	500000064119	04/27/16
03	<i>Updated BD logo and fonts to standards. Reformatted layout to comply with brand standards. Added Ukraine conformity symbol. Replaced with latest Ukraine representative address. Removed the Algeria distributor details. Updated Trademark byline, copyright year, Rev. date, SAP# and web address. Updated Rev. history and Title block templates. Replaced “DO NOT AUTOCLAVE” with “Do not sterilize” in all languages. Added “folding pattern” and “holding lines” to the artwork. Removed “Hoja 1 de 2” and “Hoja 2 de 2” text from the vendor notes. Added “Vendor Notes” text. Moved bullet point(●) for Arabic content from left to right.</i>	500000101663	08/01/17
04	<i>Updated to full “B space”. Updated symbols to symbol library. Added CH Rep and Importer symbols and addresses for Switzerland. Updated bd.com to bold. Updated trademark byline. Updated Rev. date. Added SAP #. Updated BD nomenclature and title block template.</i>	500000254294	09/27/21

en OBSERVE UNIVERSAL PRECAUTIONS ON ALL PATIENTS
REPORT NEEDLE STICKS IMMEDIATELY AND FOLLOW ESTABLISHED PROTOCOL.
•Percutaneous puncture with a contaminated needle may lead to serious illness such as hepatitis, HIV (AIDS), or other infectious diseases.
•Resheathing needles is hazardous.
NON-PYROGENIC - STERILE
Unless package has been opened or damaged.
DO NOT RESTERILIZE.
As with any spinal procedure, to help avoid needle breakage, DO NOT ATTEMPT TO STRAIGHTEN A BENT NEEDLE.
Care should be taken to avoid needle damage. Repeated repositioning may increase the risk of needle breakage. Observe needle carefully during procedures.
Remove spinal needle and introduce needle together. Replace with a new needle.
ENSURE SYRINGE IS SEATED FIRMLY IN NEEDLE HUB.
To be used by properly qualified and trained health professionals.
CAUTION: Federal (U.S.A) law restricts this device to sale by or on the order of a physician. Re-use may lead to infection or other illness/injury.

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