
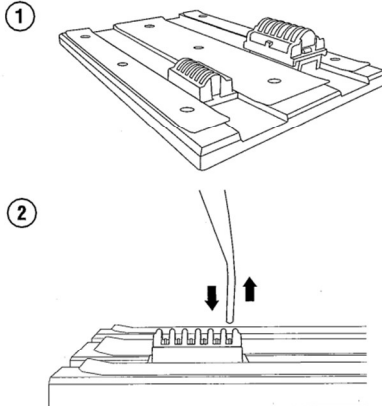
**LIGACLIP® EXTRA**

Ligating Clips  
Clips de Ligature  
Ligaturclips  
Clip di legatura  
Clips de Laqueação  
Clips de ligadura  
Ligierclips  
Ligerende clips  
Ligerausklipsit  
Κλιπ Απολίνωσης  
Ligeringsclips  
Klipsy do podwiązania  
Ligaciós csipesz  
Podvazovací svorky  
Podvazovací svorky  
结扎夹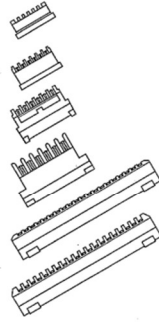

Please read all information carefully.  
Failure to properly follow the instructions may lead to serious surgical consequences, such as failure to ligate.  
Important: This package insert is designed to provide instructions for use of the LIGACLIP® EXTRA Ligating Clips. It is not a reference to ligation techniques.  
LIGACLIP is a trademark of Ethicon Endo-Surgery.

**ETHICON ENDO-SURGERY, LLC**  
a Johnson & Johnson company

Instructions, Instruções, Gebrauchsanweisung, Istruzioni, Instrucciones, Instruções, Gebrauchsanweisung, Brugsvejledning, Ohje, Ögylet, Bruksanvisning, Instruções, Utasítások, Návod k použití, Návod, 使用说明

**Indications**  
LIGACLIP EXTRA Ligating Clips have application for use on tubular structures or vessels wherever metal ligating clips are indicated. The tissue being ligated should be consistent with the size of the clip.

**Contraindications**  
• DO NOT use the instrument for contraceptive tubal occlusion.  
• DO NOT use the instrument on tissue structures or vessels upon which metal ligating clips would not normally be used.

**Device Description**  
LIGACLIP EXTRA Ligating Clips are sterile, single patient use clips produced from titanium. The clips are designed for the ligation of tubular structures where a nonabsorbable ligating device is indicated.

The ligating clips are positioned around a tubular structure and closed by applying pressure to the handles of the applicator.

**MR Conditional**  
Non-clinical testing has demonstrated the implantable clips made of titanium in these devices are MR Conditional. A patient with the implanted clips can be scanned safely immediately after placement of the clips, under the following conditions:  
- Static magnetic field of 3.0 Tesla or less  
- Highest spatial magnetic gradient field of 6.5 Tesla/m  
- Maximum MR System reported, whole body averaged specific absorption rate (SAR) of 1.2 W/kg for 20 minutes of scanning (per pulse sequence).

**MRI Related Heating**  
In non-clinical testing, a clip produced a temperature rise of less than 2°C using the following conditions:  
- At 3-Tesla (Magnetom Trio Siemens Medical Solutions MR scanner, software version Namaris/ 4 syngo MRA3.0), a maximum MR system-reported whole body averaged SAR of 1.2 W/kg  
- 20 minutes of continuous MR scanning (per pulse sequence) using transmit/receive RF body coil.

**Artifacts Information**  
MR image quality may be compromised if the area of interest is in the same area or relatively close to the position of the clips. Therefore, optimization of MR imaging parameters to compensate for the presence of the clips may be necessary.

The worst case signal void size for a clip was:

Pulse Sequence	SE	SE	GRE	GRE
Plane Orientation	Parallel	Perpendicular	Parallel	Perpendicular
Signal Void Size (mm <sup>3</sup> )	199	336	378	348

**Instructions for Use**  
Verify compatibility of all instruments and accessories prior to using the device (refer to Warnings and Precautions).  
1 Using sterile technique, remove the device from the package. To avoid damage, do not flip the device into the sterile field.  
2 Select the appropriate size LIGACLIP EXTRA Ligating Clip cartridge and corresponding clip applicator.  
3 With the cartridge slots facing away from the base, insert the cartridge into one of the channel openings of the LC80 stainless steel cartridge base. Ensure that the cartridge is part the channel opening and securely held in place (Illustration 1).

**MR**  
MR (Magnetic Resonance) Conditional  
Cautelați dacă există echipamente de rezonanță magnetică  
Bedingt MR-Magnetresonanz) -ahter  
Condição para utilização em ressonância magnética  
Uso condicionado ao ambiente de RM (Resonância magnética)  
Condição para uso EMV (resonância magnética)  
MR-voorwaarde (MR = magnetische resonantie)  
MR-tilbudet (magnetisk resonans)  
MR (magnetická rezonance) -ahter  
Aparatelor de MR (scenariu magnetic rezonanță), apă fierdă  
MR (magnetická rezonance) -ahter under vissa förhållanden  
Wskazania korzystania z obrazowania rezonansu magnetycznego  
MR (magnetická rezonance) -ahter pod určitými podmienkami  
Podmienky používania pri MRI (MR Conditional)  
Podmienky pouzitia pri rezonancii magneticko  
MR 磁共振条件

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