# CLIP HEMOSTATIQUE EN TITANE VITALITEC®

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# TITANIUM HEMOSTATIC CLIP VITALITEC®

## INSTRUCTIONS FOR USE

### PLEASE READ ALL INFORMATION CAREFULLY

Failure to properly follow the instructions may lead to serious surgical consequences.

**IMPORTANT :** This package insert is designed to provide instructions for the use of hemostatic clips. It is not a reference to surgical techniques.

- |                                  |                               |  |
|----------------------------------|-------------------------------|--|
| Medical device                   | Caution                       | Manufacturer                               |
| Batch code                       | Use by date (YYYY-MM-DD)      | Sterilized using ethylene oxide            |
| Date of manufacture (YYYY-MM-DD) | Unique Device Identifier      | Do not re-sterilize                        |
| Keep dry                         | Single sterile barrier system | Do not reuse                               |
| Consult instructions for use     | Catalogue number              | Do not use if package is opened or damaged |

CE Mark Product conforms to the essential requirements of the European Medical Devices Directive 93/42/EEC. (for USA): CAUTION: US Federal Law restricts this device to prescription only.

**CONTENTS and COLOR CODE** This box contains according to the product reference in use:

	Color code	Ref.	Contents / Blister	Ref.	Contents / Pack
<b>Range SLS</b>	White	W6060-I	30 Cartridges of 6 Micro Clips		
	Yellow	J1180-I	30 Cartridges of 6 Small Clips	J1120-I	5 Pouches of 24 Small Clips
	Red	R1180-I	30 Cartridges of 6 Small Clips	R1120-I	5 Pouches of 24 Small Clips
	Lilac	L5180-I	30 Cartridges of 6 Small/Medium Clips		
	Blue	B2180-I	30 Cartridges of 6 Medium Clips	B2120-I	5 Pouches of 24 Medium Clips
	Green	V3120-I	20 Cartridges of 6 Medium/Large Clips		
<b>Range 9</b>	Orange	O4120-I	20 Cartridges of 6 Large Clips		
	White	W9060	20 Cartridges of 9 Micro Clips		
	Yellow	J9180	20 Cartridges of 9 Small Clips	J9324	12 Blisters of 27 Small Clips
	Red	R9180	20 Cartridges of 9 Small Clips	R9324	12 Blisters of 27 Small Clips
	Lilac	L9180	20 Cartridges of 9 Small/Medium Clips		
	Blue	B9180	20 Cartridges of 9 Medium Clips	B9324	12 Blisters of 27 Medium Clips
	Green	V9120	20 Cartridges of 6 Medium/Large Clips		
	Orange	O9120	20 Cartridges of 6 Large Clips		

### INDICATIONS

The titanium hemostatic clip is intended to be used by clinically trained surgical staff. It can be used in every surgical specialty and for any patient when complete occlusion of a vessel or tissue is required, except for Fallopian Tube Ligation. The titanium hemostatic clip is made of biocompatible pure Grade 1 titanium in compliance with ISO 5832-2. It has been specifically designed to securely grip the vessel or tissue to provide complete occlusion atraumatically. User shall select appropriate clip size according to procedure ensuring that all tissues are completely encompassed by the clip. No known side effects are associated with the use of the titanium hemostatic clip. The titanium clip is biologically inert and can remain in vivo without sequelae.

**MR Information:** non-clinical testing demonstrated that the titanium hemostatic clip is MR conditional. A patient with this device can be scanned safely immediately after placement under the following conditions:

- Static magnetic field of 3 Tesla or less;
  - Maximum spatial gradient magnetic field of 720 Gauss/cm;
  - Maximum MR system reported, whole body averaged specific absorption rate (SAR) of 4 W/kg for 15 minutes of scanning (i.e., per pulse sequence).
  - First Level Controlled Operating Mode of operation for the MR system.
- In non-clinical testing, the titanium hemostatic clip produced a maximum temperature rise of 1.8°C during MRI performed for 15 minutes of scanning on the 3 Tesla MR system.
- MR image quality may be compromised if the area of interest is in the exact same area or relatively close to the position of the titanium hemostatic clip. The maximum artefact size (i.e., as seen on the gradient echo pulse sequence) extends approximately 5 mm relative to the size and shape of the titanium hemostatic clip.

### CONTRAINDICATIONS

This clip is not intended for Fallopian Tube Ligation.

### INSTRUCTIONS

- Inspect the integrity of the individual sterility protection (packaging) before using. Do not use if sterile packaging is opened or damaged.
- Open the sterile packaging.
- Remove the protective strip from the adhesive base of the clip cartridge and press firmly on a stable surface.
- Select VITALITEC applicator according to clip size (color coding).
- Hold applicator at the box lock.
- Introduce the jaws of the applicator vertically into the cartridge clip housing.
- Push them through the housing to the bottom of the cartridge for an accurate clip loading.
- Place the loaded clip and jaws of the applicator around the vessel or tissue to be occluded and firmly squeeze the handles together closing the clip. First, the clip closes tip to tip preventing vessel slippage before fully closing.
- Release the handle and remove applicator.

### WARNINGS

**To prevent premature closure or release of the clip:**

- Never grasp the applicator by both shanks when loading the clip.
  - Do not squeeze applicator handles before jaws are placed around vessel or tissue.
  - Apply a force necessary to ensure applicator jaws are fully closed, until the handle limit stop.
- Inspect ligation site to ensure secure clip application and ensure that hemostasis is performed, after vessel resection.**
- To prevent damage to applicator jaws, avoid introducing applicator tips at an angle into the cartridge housing, this may misalign tips.**
- Inspect applicator and jaws before each use.**
- Use exclusively applicators for SLS or «9» Clips supplied by Péters SURGICAL.**

### STORAGE

Store the unopened sterile package in a cool dry area, with good ventilation away from areas of extreme temperature and humidity.

### SHELF LIFE

Contents sterile if package is unopened, undamaged and within the validity date printed on the package. **THIS CLIP IS INTENDED FOR SINGLE USE AND CANNOT BE RESTERILIZED ONCE THE PACKAGE IS OPENED.** Do not reuse, reprocess, or re-sterilize. Reusing, reprocessing or re-sterilization may compromise the structural integrity of the device and/or lead to device failure, which in turn may result in patient injury, illness or death. Reusing, reprocessing or re-sterilization may also create a risk of contamination of the device and/or cause infection including but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient. Check the packaging integrity before use. Sterility guaranteed unless package opened or damaged. The manufacturer declines responsibility for problems resulting from improper use.

### DISPOSAL

The titanium hemostatic clip does not contain any hazardous substance. It is disposed of as per the local or national hospital waste disposal management procedures.