

Product code C222-305

PRODUCT SPECIFICATION

Product Range		PLAIN FAST EDGE RIBBON GAUZE	
Product Name		Plain Fast Edge Ribbon Gauze 1.25cm x 5m	
Product Code	Product Description	Size / Dimensions / Tolerances	Material / Composition / Active Ingredients
03820	Plain Fast Edge Ribbon gauze 1.25cm x 5m Non-sterile White	1.25cm± 0.3cm 5m± 40cm Ø3.8cm ±0.5cm	100% Cotton yarn
Intended Use		The device is intended to be used in packing cavities which have breached the dermis and require healing by secondary intent.	
Classification		Class IIb Rule 4	
Product/Device Single Use		Yes	
Components Latex Free		Yes	
pH Information		N/A	
Shelf Life		5 years form the date of manufacture	
Storage		Cool dry environment away from direct sunlight	
Sterilisation Method		Non-sterile	
Special Instructions			
<ul style="list-style-type: none"> - Device is supplied non-sterile and is intended to be sterilised before use. - Device is suitable for one sterilisation cycle using Steam sterilisation, Ethylene oxide or Gamma Irradiation. - Device is single use. - Once sterile the device should not be re-sterilised. - Device is intended to be used by medical personnel. - When used in cavity wounds and fistulas, always leave a part of the dressing protruding, this will aid external location and removal. - Dressing change frequency will be based on Clinicians discretion, wound condition and the level of exudate. 			